

Chirana

Recommended Use Hypodermic Syringe for Single Use



STERILE EO





- Description:** Hypodermic syringes for single use. Sterile, Non-toxic, Non-pyrogenic.
Intended use for: Injection of fluids into body and withdrawal of blood and fluids from the body
 For FNA and FNNA procedures we recommend to use 3-part syringes CHIRANA.
Contraindication: Medical device should not be used other than as indicated in intended use.
 Medical device should not be used for high viscous fluid.



1. Before usage confirm the sterility validity, packing completeness, the LOT number and expiry date;
2. The syringes should be used immediately after unit package opened, discard after use;
3. It is not allowed for use if unit package damaged and validity expired;
4. To be used by qualified doctor or paramedic only;
5. Only for single use. Discard the set after single use;
6. This medical device does not contain phthalates DEHP;
7. Re-sterilization or re-use could cause infection or cross-interactions.
8. If after filling the syringe the solution becomes cloudy or discoloration or particles occur, immediately stop use, inform manufacturer and if possible return the syringe to the manufacturer.



1. Examine individual package for integrity of packaging prior to use. If packaging has been torn, do not use;
2. Before usage, open the unit package;
3. Any used syringe is considered contaminated and should be disposed of in an approved container immediately after use;
4. Dispose of safely after single use to avoid risk of infection.

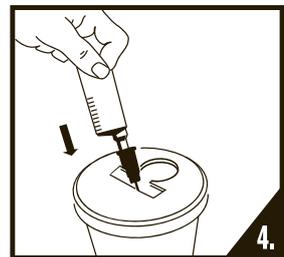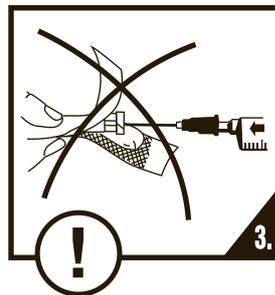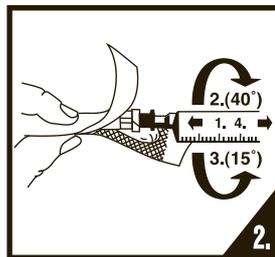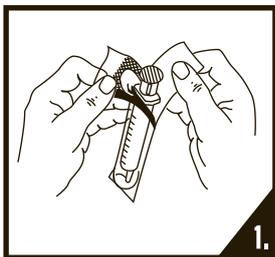

Caution



Consult instructions for use



Serial number



Do not reuse



Batch code



Date of manufacture



Used by

STERILE EO

Sterilized by Ethylene Oxide



Manufacturer



Pyrogen free



LATEX free



Do not use if package is damaged



Keep away from sunlight



Temperature limitation



Keep dry



Fragile, Handle with care

