

# Instructions for use

**PRODUCT NAME:** **INFUSION SET FOR SINGLE USE**

**BRAND NAME:** **CHIRAPLUS - P / CHIRAPLUS - G**

**INTENDED PURPOSE:**

- Infusion set for single use is intended for administration liquids and drugs into the circulation system by using of intravenous catheters and cannulas.

**INTENDED USERS:**

- Health care professionals with relevant knowledge of use these types of medical devices. Infusion sets are simple and well-known products.

**INTENDED PATIENT POPULATION:**

- No specific requirements or restrictions on use for patient population or user group defined by the manufacturer.

**CONTRAINDICATIONS:**


- Not to be used other than as indicated in intended purpose.
- Administration of highly viscous fluids.
- Blood Transfusion.

**INSTRUCTIONS FOR USE:**

- Remove Infusion set from unit package, closed the roller clamp with flow regulator.
- Prepare container with infusion fluid.
- Remove protector from piercing spike. Insert the spike to its full length into the top of the Infusion fluid container. Suspend/hang the container.
- Squeeze the drip chamber and set fluid level.
- Hang the connector end in the groove provided in the roller clamp.
- Open the airvent cap of piercing spike.
- Open the clamp. The fluid starts running into the tubing and priming takes automatically up to the connector end. Close the clamp.
- Remove protective cap from the connector end and connect with venous access device.
- Open roller clamp and adjust flow of infusion with flow regulator.

**CAUTIONS:**

- Read instructions before use. The product should be used according to the Instructions for use.
- Check the product and package integrity prior to use and also the expiry date.
- Do not use if protective cap is loose or missing.
- Do not use if package is opened or damaged.
- For pressure or gravity feed, as marked on product packaging (G = Gravity, P=Pressure).
- Use the product immediately after opening the individual blister packing.
- Close the airvent during periods of interrupted infusion therapy.
- Do not re-sterilize. Discard the set after single use.
- This device does not contain phthalates (**DEHP FREE**); as marked with symbol .
- Infusion set with Flashball contains **LATEX** and is marked with symbol .
- Reuse and cleaning of the product may alter their structural and mechanical properties. It may lead to infections or other illness/injury.
- The device should be replaced and dispose as per facility approved protocol or guidelines.
- Store at 0-40°C temperature, avoid excessive heat, protect from direct sunlight and moisture.
- **CHIRANA T. Injecta** will not be responsible for any direct incident or consequential damages resulting from reuse or improper use of the product.
- Report any health incident associated with medical device to healthcare professional or manufacturer.

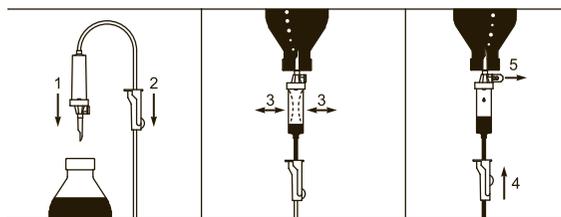

**CHIRANA T. Injecta, a.s.**

Nám. Dr. Schweitzera 194  
916 01 Stará Turá  
Slovak Republic  
Tel.: +421/32/7709 991

[www.t-injecta.com](http://www.t-injecta.com)



-  Fragile, handle with care
-  Keep away from sunlight
-  Keep dry
-  This product meets the requirements of the Regulation 2017/745

-  Caution
-  Consult instructions for use
-  Do not re-use
-  Do not re-sterilize
-  Do not use if package is damaged
-  Catalogue number
-  Batch code
-  Date of manufacture
-  Use-by date
-  Sterilized using ethylene oxide
-  Single sterile barrier system
-  Single sterile barrier system with protective packaging outside
-  Distributor
-  Temperature limit
-  Manufacturer
-  Non-pyrogenic
-  DEHP Free
-  Contains or presence of natural rubber latex
-  LATEX Free
-  Liquid filter with pore size
-  Drops per millilitre
-  Unique device identifier
-  Medical device