

CHIRAPLUS

Instruction for Use Infusion set for Single Use



- Description:** Infusion set for single use. Sterile, Non-toxic, Non-pyrogenic.
- Components:** The products composed of cap of closure piercing device, closure piercing device, tubing, drip chamber, drip tube, blood filter, flow regulator, scalp vein set etc.
- Application scope:** Used for clinical venous infusion, and taking part in medical treatment.
- Prohibition:** Do not use for blood transfusion and sticky solution.




1. Before usage, confirm the sterility validity, packing completeness, the lot No. and expiry date are in order;
2. The products should be used immediately after envelope opened, discard after use;
3. It is not allowed for use if envelope damaged, cap falling off and validity expired;
4. If there is unusual cased during infusion, treat according to the clinical operation procedures;
5. This device contains phthalates DEHP;
6. The infusion of lipid based solution could increase the risk of endocrin disruption in children, infants or pregnant women should be avoided;
7. Re-sterilization or re-use could cause infection or cross-interactions.

8. Infusion set with Flashball contains LATEX and is marked with symbol .



1. Before usage, open the envelope from the „open here“ labeled place;
2. Close the flow regulator, take off the cap from the closure piercing device, insert the closure piercing device into the infusion bag and keep sure of no leakage;
3. Press the drip chamber with finger, and let the infusion solution fill half of the chamber;
4. Open the flow regulator, drain the air, observe the dripping, and start the infusion;
5. Before usage, insert the needle tightly and avoid leakage.

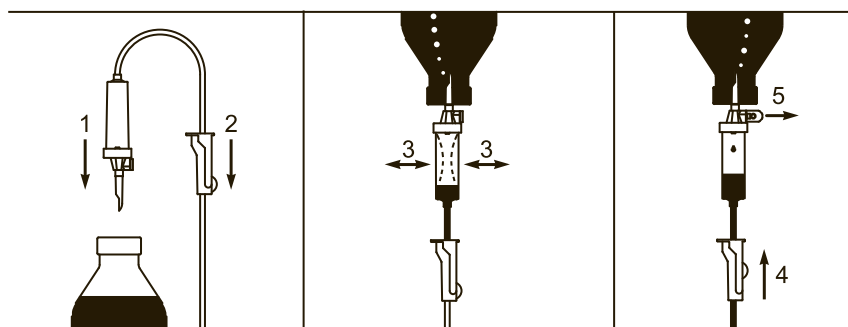

Attention



See instruction for use



Serial number



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Manufactured by



For single use only



LATEX Free product



With content or presence of natural rubber LATEX



Do not use if unit package is damaged.



Pyrogen free



Filter of liquid with pore size



The number of drops per ml



Keep away from sunlight.



Temperature Limitation.



Keep away from Rain.

CHIRANA T. Injecta, a.s.
 Nám. Dr. Schweitzera 194
 916 01 Stará Turá
 Slovak Republic
 Tel.: +421/32/7709 991
 Fax.: +421/32/7709 956
www.t-injecta.com