

# INSTRUCTIONS FOR USE

## CHIRAWAY Three Way Stop Cock

### MATERIALS USED:

Polypropylene, Polycarbonate, HDPE.

### INDICATIONS:

- To administer two drugs or fluids at the same time.

### CONTRAINDICATIONS:

- Administration of highly viscous fluids.
- Blood transfusion.
- Not to be used in patients with known hypersensitivity to any of the materials used.

### INSTRUCTIONS FOR USE:

1. Connect I.V. infusion set to one or two port of three way stop cock after removing white cap (luer lock).
2. Open clamps of I.V. infusion lines and expel the air from three way stop cock.
3. Arrow indication on handle indicates direction of flow and can administer two fluids or drugs at the same time.
4. Rotate the handle to close position.
5. Connect the male luer to venipuncture device by applying a slight torque along with an axial force, now engage the luer lock assembly by applying a rotating action on luer lock.
6. Rotate the handle to open position and adjust the flow of infusion set.

### CAUTIONS:

- Do not use if package is opened or damaged.
- Use three way stop cock immediately after opening the individual blister packing.
- The product should be used according to the Instructions for use.
- Some organic solvents, alcoholic disinfectants, infusion solutions and high pH-value substances might cause the stopcocks material to crack.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- CHIRANA T. Injecta will not be responsible for any direct incident or consequential damages resulting from reuse of product.

### WARNING

- Lipids can be used only with Chiraway Pro/Chiraway pro-Click.

### CHIRANA T. Injecta, a.s.

Nám. Dr. Schweitzera 194  
916 01 Stará Turá  
Slovak Republic  
Tel.: +421/32/7709 991  
FAX: +421/32/7709 956

 [www.f-injecta.com](http://www.f-injecta.com)

  
2460

  
CHIRANA  
T. Injecta



Attention



See instruction for use



Product reference/Art. No.



For single use only



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Manufactured by



LATEX Free product



DEHP Free product



Do not use if unit package is damaged



Non pyrogenic.



Keep away from sunlight.



Temperature Limitation.



Keep away from Rain.