


CHIRAHEM

Instruction for Use Blood Transfusion set for Single Use




- Description:** Blood transfusion set for single use. Sterile, Non-toxic, Non-pyrogenic.
- Components:** The products composed of cap of closure piercing device, closure piercing device, tubing, drip chamber, drip tube, blood filter, flow regulator, etc.
Different gauges of needle can be optioned.
- Application scope:** Used for clinical blood transfusion, and taking part in medical treatment.
- Prohibition:** Used only for blood transfusion.









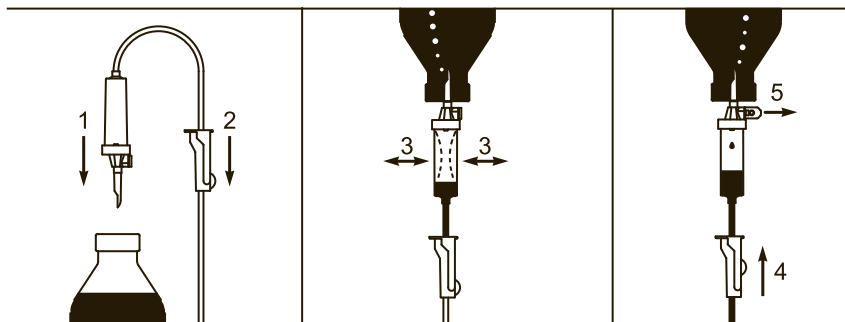






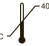




- Before usage, confirm the sterility validity, packing completeness, the lot No. and expiry date are in order;
- The products should be used immediately after envelope opened, discard after use;
- It is not allowed for use if envelope damaged, cap falling off and validity expired;
- If there is unusual cased during transfusion, treat according to the clinical operation procedures;
- This device contains phthalates DEHP;
- Longterm use of blood transfusion sets containing DEHP with infants, children or pregnant women, could cause an increased risk of endocrine disruption;
- Re-sterilization or re-use could cause infection or cross-interactions;
- Transfusion set with Flashball contains LATEX and is marked with symbol .



- Before usage, open the envelope from the „open here“ labeled place;
- Close the flow regulator, take off the cap from the closure piercing device, insert the closure piercing device into the blood bag and keep sure of no leakage;
- Press the drip chamber with finger, and let the blood fill half of the chamber;
- Open the flow regulator, drain the air, observe the dripping, and start the transfusion;
- Before usage, insert the needle tightly and avoid leakage.




CHIRANA T. Injecta, a.s.
 Nám. Dr. Schweitzera 194
 916 01 Stará Turá
 Slovak Republic
 Tel.: +421/32/7709 991
 Fax.: +421/32/7709 956
www.t-injecta.com

	Attention		Do not use if unit package is damaged.
	See instruction for use		Pyrogen free
	Serial number		Filter of liquid with pore size
	Batch Number		The number of drops per ml
	Date of Manufacturing		Keep away from sunlight.
	Use by / Expiry date		Temperature Limitation.
	Sterilised by Ethylene Oxide		Keep away from Rain.
	Manufactured by		
	For single use only		
	LATEX Free product		
	With content or presence of natural rubber LATEX		